

STORAGE AND STABILITY:

STORE UNOPENED CONTROL IN REFRIGERATOR (2-8 c). CONTROLS ARE STABLE UNTIL EXPIRATION DATES SHOWN ON LABELS.

RECONSTITUTED TOTAL HGB CONTROLS ARE STABLE FOR 7 DAYS STORED AT 2-8C.

INDICATION OF DETERIORATION:

IF THE PRODUCT APPEARS BROWN AND/OR FORM PRECIPITATE WHEN ADDED TO CYANMETHEMOGLOBIN REAGENT, DO NOT USE THE PRODUCT.

DIRECTIONS:



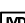





RECONSTITUTION: ADD 3.0 ML OF REAGENT GRADE PURIFIED DISTILLED WATER TO CONTENTS OF ONE VIAL. MIX GENTLY FOR 2-3 MINUTES. DO NOT SHAKE. LET STAND FOR 10 MINUTES BEFORE USING. BEFORE USE EXAMINE PRODUCT TO MAKE SURE THAT ALL CONTENT IS WELL RECONSTITUTED. SWIRL AGAIN IF NEEDED.

APPLICATION:
USE SAMPLE CONTROL AS YOU WOULD WHOLE BLOOD.

TREAT THIS SAMPLE THE SAME AS RECOMMENDED FOR PATIENT SAMPLE. PRECISION AND ACCURACY IS DRAMATICALLY AFFECTED BY TECHNIQUE. MAKE SURE ALL PROPER PIPET TECHNIQUES ARE FOLLOWED AS TECHNIQUE AFFECTS ACCURACY AND PRECISION.

SAFETY AND PRECAUTIONS:

FOR IN VITRO DIAGNOSTIC USE. ALL BLOOD PRODUCTS SHOULD BE TREATED AS POTENTIALLY INFECTIOUS. SOURCE MATERIAL FROM WHICH THIS PRODUCT WAS DERIVED IS FROM NON-HUMAN SOURCES.

	Lagertemperatur		Chargenbezeichnung
	Nur für in-vitro diagnostische Zwecke		Verwendbar bis
	Bestell-Nr.		Inhalt ausreichend für <n> Tests
	Gebrauchsanweisung beachten		Hersteller

	 BGT BioGenTechnologies GmbH Von-Langen-Weg 10 D-48565 Steinfurt Tel.: 02551/4090 Fax.: 02551/1298	
---	--	---