

Art.-Nr.: KG1368
Ch.-B.: 3351MS
Inhalt: 6 x 2 ml
Verw. bis: 2009-06

INTENDED USE

This product is intended for *in vitro* diagnostic use in the quality control of Ammonia and Alcohol Assays on clinical chemistry systems. The Ammonia Ethanol Controls are for the control of accuracy and precision.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Human source material from which this product has been derived has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

UNOPENED: Material is stable to expiry date when stored at +2°C to +8°C if kept capped in original container and free from contamination.

OPENED: Material is stable for 30 days in opened vials. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

PREPARATION FOR USE

The Ammonia Ethanol controls are supplied ready for use.

MATERIALS PROVIDED

Ammonia Ethanol Control Level 3 6 x 2ml

ASSIGNED VALUES

Values are assigned for each batch of Alcohol ethanol controls internally at BGT Laboratories Limited.

ANALYTE	TARGET	RANGE
AMMONIA (µmol/l)	372	298 - 446
ETHANOL (g/l)	2.73	2.18 - 3.28

30 July '08