

PREPARATION

Carefully reconstitute each vial of lyophilised CSF with exactly **3 ml** of distilled water. Close bottle and allow to stand for 10 mins before use. Ensure contents are completely dissolved by swirling gently. Avoid the formation of foam by shaking.

STABILITY

The lyophilised CSF is stable until the expiry date when stored at +2 to +8°C. The components of the reconstituted serum are stable for 2 days at +2 to +8°C.

SAFETY

The CSF is of human origin. Each batch has been tested and found negative for HIV (HTLV III) antibodies and hepatitis B surface antigen. Nevertheless, the control should be handled as if capable for transmitting infection and disposed of accordingly.

FOR IN VITRO USE ONLY.

Parameter	Methode	Zielwert	Bereich	Einheiten
Chloride	I.S.E.	85	72 - 98	mmol/l
Glucose	GOD/PAP	2.80	2.46 - 3.14	mmol/l
Lactic Acid	Enzymatic	0.39	0.33 - 0.45	mmol/l
Sodium	I.S.E.	130	111 - 150	mmol/l
Total Protein	Pyrogallol Red	0.29	0.25 - 0.33	g/l
Total Protein	Coomassie Blue	0.24	0.20 - 0.28	g/l
IgG	Beckman Immage	41.0	34.9 - 47.2	mg/L

Protein Electrophoresis	% Total Protein
Albumin	55.4
Alpha - 1 Globulin	6.2
Alpha - 2 Globulin	10.0
Beta Globulin	13.2
Gamma Globulin	15.2