

Art.-Nr.: KG3444
Ch.-B.: 1051HA – 1055HA
Inhalt: 1 x 8 ml + 5 x 2 ml
Verw. bis: 2010-08

INTENDED USE

This product is intended for *in-vitro* diagnostic use in the calibration of HbA_{1c} on clinical chemistry systems.

SAFETY PRECAUTIONS AND WARNINGS

For *in-vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

The calibrator must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions. Avoid contact with eyes, skin & clothing. Wash thoroughly after handling. Avoid ingestion.

Calibrator Level 1 Irritant. Alkaline Solution containing 0.4% NaOH.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

Opened: Store refrigerated (2°C to 8°C). Stable to expiration date if kept capped in original containers and free of contamination.

Unopened: Store refrigerated (2°C to 8°C). Stable to expiration date printed on the individual vials.

PREPARATION FOR USE

1. Mix by inversion at least 5 times to ensure homogeneity prior to use.
2. Refer to the calibration section of the individual analyser application.
3. Refrigerate any unused material. Prior to reuse, mix contents thoroughly.

NB. Calibrators Do Not Require Pretreatment

MATERIALS PROVIDED

Haemoglobin A_{1c} Calibrator Level 1 (1 x 8 ml)
Haemoglobin A_{1c} Calibrator Level 2 (1 x 2 ml)
Haemoglobin A_{1c} Calibrator Level 3 (1 x 2 ml)
Haemoglobin A_{1c} Calibrator Level 4 (1 x 2 ml)
Haemoglobin A_{1c} Calibrator Level 5 (1 x 2 ml)
Haemoglobin A_{1c} Calibrator Level 6 (1 x 2 ml)

MATERIALS REQUIRED BUT NOT PROVIDED

None

ASSIGNED VALUES

A value (mean) has been assigned to this lot of calibrator at BGT Laboratories Ltd.

LEVEL	SIZE	PROD. NO	LOT. NO.	EXP	HbA _{1c} g/dl	TOTAL HAEMOGLOBIN g/dl
1	8 ml	HA022	1051HA	2010-08	0.00	18.0
2	2 ml	HA023	1052HA	2010-08	0.47	N/A
3	2 ml	HA024	1053HA	2010-08	0.78	N/A
4	2 ml	HA025	1054HA	2010-08	0.99	N/A
5	2 ml	HA026	1055HA	2010-08	1.54	N/A
6	2 ml	HA027	1056HA	2010-08	2.06	N/A

Calibrator Values have been assigned as a consequence of NGSP traceability on the Dimension analyzer. A value (mean) has been assigned to this lot of calibrator at BGT Laboratories Ltd.

LEVEL	SIZE	PROD. NO	LOT. NO.	EXP	HbA _{1c} g/dl	TOTAL HAEMOGLOBIN g/dl
1	8 ml	HA022	1051HA	2010-08	0.00	18.0
2	2 ml	HA023	1052HA	2010-08	0.46	N/A
3	2 ml	HA024	1053HA	2010-08	0.75	N/A
4	2 ml	HA025	1054HA	2010-08	0.95	N/A
5	2 ml	HA026	1055HA	2010-08	1.45	N/A
6	2 ml	HA027	1056HA	2010-08	2.30	N/A

27 Jan '09 ne